
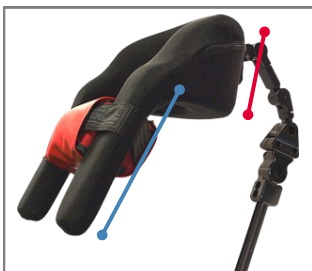


TECHNICAL DATA

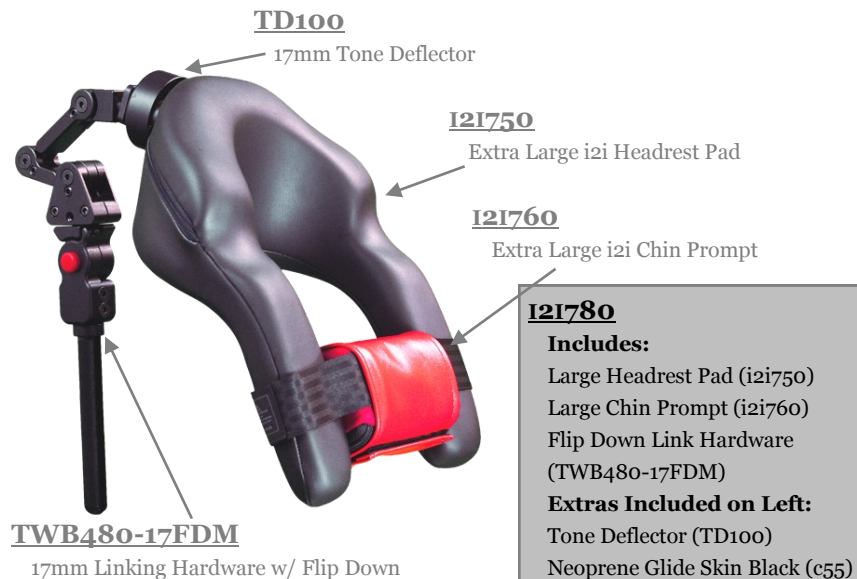

DIMENSIONS

Product Number	Overall Width	Maximum Neck Circumference	Maximum Neck Width	Posterior Height	Arm Length
i2i 280 Niño X-Small	7"	7"	3"	3"	4½"
i2i 380 Niño Small	7½"	9"	3½"	3½"	4½"
i2i 480 Small Standard	8"	11"	4"	4"	6½"
i2i 481 Small Short Arm	8"	11"	4"	4"	4½"
i2i 580 Medium Standard	9"	14"	4½"	4¾"	7½"
i2i 581 Medium Short Arm	9"	14"	4½"	4¾"	5½"
i2i 680 Large Standard	9½"	16"	5"	5¼"	8"
i2i 680 Large Short Arm	9½"	16"	5"	5¼"	6"
i2i 780 X-Large Standard	10"	18"	5¾"	6"	9"
i2i 781 X-Large Standard	10"	18"	5¾"	6"	6¼"

PRODUCT DESCRIPTION

The i2i has been one of Stealth's signature products and one that is recognized around the world. The i2i head, neck and shoulder positioning system is offered in eleven different series. We have made significant improvements through the years and now have added a few more. It's important to remember that the Headrest is the finishing point of any seating system—not a starting point. If a client does not hold his/her head erect it does not necessarily mean he has poor head control. *It could be the result of a poorly fitting or improper seating system!*

TECHNICAL DATA

1. 5/32 tool for socket-head and flat-head screws on mounting ring and mounting hardware.
2. Includes: ball mounting ring and headrest pad.
3. Headrest attaches to the TWB480-FDM.

SELLING POINTS

1. The i2i is designed to help those with difficulty of movement and keeping the head upright.
2. Keeping the head straight and level improves eye contact to promote socialization and assist in interpreting responses and communication.
3. Keeping the user's head straight eases feeding, reduces saliva drool and greatly reduces the need to reposition the user's head, neck, and shoulders.

BUILD YOUR i2i SERIES HEADREST

STEP 1: NECK SIZE

The individual's neck circumference will determine the appropriate size i2i.

NECK SIZES

480	Under 11"
580	11"-14"
680	14"-16"
780	16"-18"

STEP 2: ARM LENGTH

To determine arm length, measure from the top of the clavicle to the bottom of the sternum.

ARM LENGTHS

i2i Standard

480	6.5"
580	7.5"
680	8"
780	9"

i2i Short Arm

481	4.5"
581	5.5"
681	6"
781	6.25"

STEP 3: EAR CUTOUTS (i2i-EC)

Individuals with lateral flexion will need ear cutouts and gel inserts to help reduce friction and pressure points.

STEP 4: FABRICS

Select a material for both the i2i and the Chin Prompt.

FOAM/GEL

Closed Cell Foam
Action Gel

COVERS

Dartex (standard)
Reverse Dartex
Vinyl
Sheep Skin
Neoprene

STEP 5: LINKS (TWB-LINK)

Standard i2i Hardware has a maximum reach of 6". Additional links increase the maximum reach.

STEP 6: SWING-AWAYS (SUS9 SERIES)

Facial lateral Swing-Aways can be added to the i2i for pad, switch, speaker sites and chin control devices.

HARDWARE OPTIONS



TWB480-17FDM

An easy transfer flip down mount and links make Stealth's TWB480-FDM.



TD100

The Tone Deflector is design to absorb quick impact and aver slanted angle loading.

i2i HEADREST PACKAGES*



i2i280
Niño Extra Small

i2i282
Standard w/Gel



i2i480
Small i2i

i2i482
Standard w/Gel



i2i490
Low Profile

i2i492
Low Profile w/Gel



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
i2i440
Adjustable Arms**

i2i442
Adjustable w/Gel



i2i580
Medium i2i

i2i582
Standard w/Gel



i2i590
Low Profile

i2i592
Low Profile w/Gel




i2i540
Adjustable Arms**

i2i542
Adjustable w/Gel



i2i680
Large i2i

i2i682
Standard w/Gel

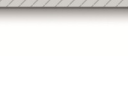

i2i690
Low Profile

i2i692
Low Profile w/Gel



i2i640
Adjustable Arms**

i2i642
Adjustable w/Gel

i2i780
Extra Large i2i

i2i782
Standard w/Gel

i2i790
Low Profile

i2i792
Low Profile w/Gel

i2i740
Adjustable Arms**

i2i742
Adjustable w/Gel




i2i380
Niño Extra Small

i2i382
Standard w/Gel



i2i481
Sm. w/short arms

i2i483
Standard w/Gel

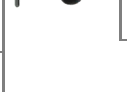

i2i491
Low Profile

i2i493
Low Profile w/Gel



i2i581
Med. w/short arms

i2i583
Standard w/Gel



i2i591
Low Profile

i2i593
Low Profile w/Gel



i2i681
Lg. w/short arms

i2i683
Standard w/Gel



i2i691
Low Profile

i2i693
Low Profile w/Gel



i2i781
X-Lg. w/short arms

i2i783
Standard w/Gel



i2i791
Low Profile

i2i793
Low Profile w/Gel

SWING-AWAY OPTIONS

More information available at www.stealthproducts.com



SUS9-B-HW

Stealth Product's Swing-Away hardware offers a sleek, unobtrusive design while featuring an easy, one-hand release mechanism for swinging pads and switches back a full 100°.



SUS9DL-DB

The dual swing away design allows for more attachment sites for facial pads or switches for those clients who need more real estate or extra support.



SUNS9-B-HW

Convenient sizing, versatility, comfort and intuitive function with a clean, elegant design makes Stealth's SUNS9 the best choice when in need for pediatric facial pads and switches.



900
Facial Pad
(4-5 " x 2 ")



910
Facial Spot Pad
(3 " x 2 ")



920
Long Facial pad
(6 " x 2 ")



930
Long Niño Facial Pad
(6 " x 1.5 ")



F8675

Stealth's figure 8 bracket adds real estate for an extra attachment.



JOYSTICK MOUNTS

SM600: Mini Proportional mount
SM610: 1812 mount
SM620: Tash Mini mount
SM630: ASL MEC mount



SP625

Sip & Puff mount



CN600

Swing-away flex Micro-Lite switch.

Stealth Products, LLC 1.800.965.9229
104 John Kelly DR. Burnet, TX 78611

*Headrest Packages come with Headrest Pad, Link Style Flip Down Mounting hardware and Chin Prompt.

**Packages with 'Adjustable Arms' and 'Adjustable w/Gel' do not include a Chin Prompt.