

Stealth Products

# Tone Deflector

the hardware that takes you over the limits!



Stealth Products introduces the Tone Deflector! This option will serve two main functions:

## Absorb and Avert

This intuitive design will absorb quick impact compression and avert slanted angle loading. The Tone Deflector (TD) will be the premier addition for any high tone situation.

Available for any single pad headrest!

[AP, Combo, Comfort, i2i, OB]



Back View

This would receive the hardware mounted to the back.



Front View

This piece would be mounted to the headrest.




Easy Disassemble  
Disassembled View.

## 10 DEGREES




The new Tone Deflector offers a 10° deflection to any direction


TOP VIEW

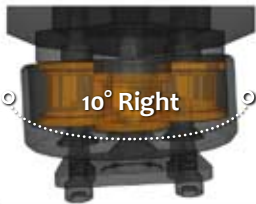

10° Right

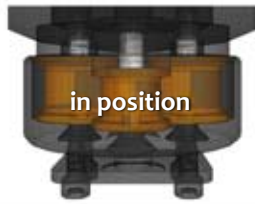


in position

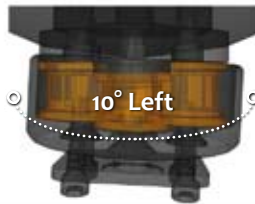


10° Left

SIDE VIEW

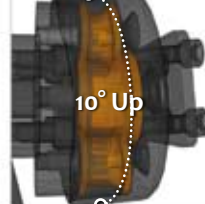

10° Up

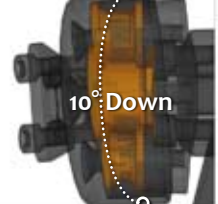


10° Down

## Assembling Guide.

Four easy steps!

- When disassembled, mount the TD adaptor plate to the back of the headrest pad.
- Mount the Tone Deflectors to the TD adaptor plate.
- Set the TD Cover to the deflectors.
- Finally, attach the steel SU Ring to the TD Cover.

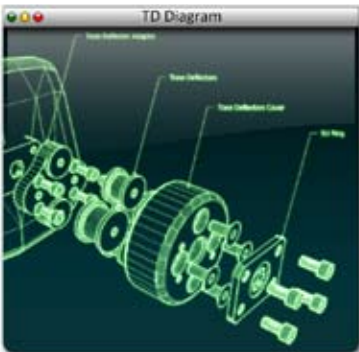

to order or more information call  
**1.800.965.9229**

This is another innovative product designed by

