

# permobil

## CORPUS 3G

SEATING SYSTEM ACCESSORIES



### HEAD SUPPORT



Comfort Plus

### LATERAL SUPPORT



Thin Laterals



TWBL Laterals

### PELVIC & KNEE SUPPORT



Knee Stops



Pelvic Support

### ACCESSORIES SUPPORT



Gatlin Mounts

POSITIONING  
PROVIDED BY

  
**Stealth  
Products**



## THE NEW CORPUS SERIES

PERMOBIL HAS RELEASED THEIR NEW CORPUS SERIES MOBILE CHAIRS AND IN TURN STEALTH PRODUCTS HAS DEVELOPED A WIDE RANGE OF MOUNTING OPTIONS FOR OUR MANY ACCESSORIES TO GIVE A CLEAN SLEEK LOOK.

### HEADREST HARDWARE

The Corpus 3G back has a 7" horizontal track for the seat back mount. This gives an adjustment of about 5" for left and right movement when positioning the headrest hardware. The track can also be moved vertically on a pre-drilled hole step pattern.

### ULTRA SERIES HEAD POSITIONING SYSTEM

The Stealth Ultra (**SU**) headrest has the widest range of adjustments for support and positioning. Swing away pads and switches can be added to give a wide range of support and functions.

### COMFORT PLUS SERIES HEADREST

For standard head support we offer our Comfort Plus (**CP**) series. This headrest can be equipped with embedded egg or proximity switches for a sleek look.

### TRI-COMFORT PLUS SERIES HEADREST


Take the standard out of the standard with the latest Stealth's headrest, the Tri-Comfort Plus series. Almost like the Combo Series, the Tri-Comfort extends with ergonomic lateral support that promote additional stability and resting points. With many options to choose from the **TCP** series is versatile and easy on the eyes.

### COMBO SERIES HEADREST

The Combo (**CB**) series is a winged style headrest that cradles the head without having additional head supports. It consists of a posterior pad and two lateral pads.

### ALL POSITIONING SERIES HEADREST

For positioning and support, the All Positioning (**AP**) is a simple but sleek headrest. Made of a very unique malleable metal, the AP can be adjusted on demand.

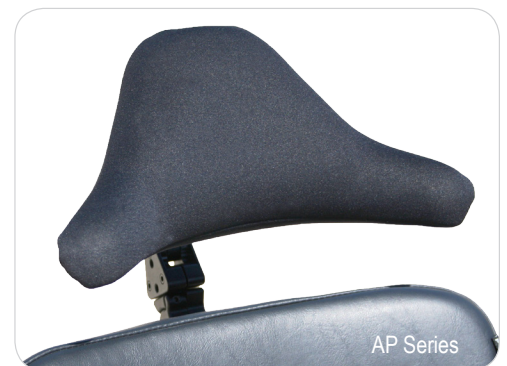



## LATERAL SUPPORT

The **Worlds Best Laterals Series (TWB)** contains the most versatile laterals on the market. The TWB series can be used for thoracic (**TWBL**) and pelvic support (**TWBADD**), and also comes in a miniaturized (**Niño Series**) pediatric version (**TWBNL**). The TWB series is comprised of link hardware, that allows for inset/offset adjustment and growth manipulation to accommodate any size client with ease. The swing-away feature is equipped on all TWB series laterals allowing for easy transfer. The addition of the Full Surface Contact (**FSC**) feature takes this lateral to a whole new level by offering support on a 360° axis. It allows for the pad to simulate therapist's hands by rotating to the curvatures of the body. If parts of the chair interfere with the swing away feature, then the **SWINK** option would give a second position for links to swing away.



Pelvic Support (TWBADD)

## THIN STYLE LATERALS

Stealth Products also offers a sleeker thin style lateral (**TWBTL**) without link or with link (**TWBSTL**) that have a slim line profile and reduced weight. The thoracic laterals mount with an after market track developed by Stealth Products for the CS3G seating, while the pelvic utilizes the tracks on the seat.



Thin Laterals (TWBSTL)

## KNEE STOPS

The stops add comfort to the body and keep limbs positioned with ease. They swing away to assist in transfers as well and adapt to the tracks that Permobil implemented on their chair (**KN1ADD-TRK**) or in their existing mounts (**KN1ADD**).



Knee Stops



Elbow Stops

## ELBOW STOPS


These support/stops feature the swing away option and the rotating pads that will move in almost any direction. They can be mounted on tracks or on canes. Many options are available for the foam and cover materials.

## PORTABLE DIGITAL MEDIA MOUNT

Stealth has developed several mounts for digital electronic peripherals. These include the iPad® (1&2), Motorola® xZoom™, laptops, and universal cell phone mounts. The TWBM260-IP is the adapter for these peripheral mounts that we have created to mount under armrests of the new Permobil chairs that are equipped with track mounting.



Stealth Tablets Mounts

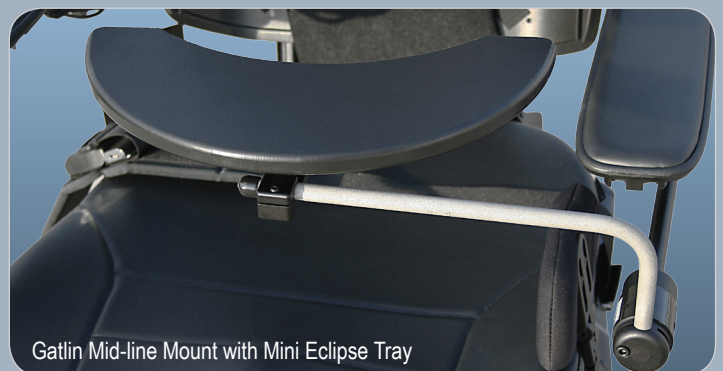

Gatlin Mid-line Mount with Mini Eclipse Tray

## GATLIN MID-LINE MOUNTS

The Gatlin series is a mid-line positioning system for trays, switches, displays, and driver controls. The GAT400P has 8 locking positions and GAT405P is a swing away style that rotates on a 95° or 180° horizontal rotation. These part numbers will mount in the arm mounts provided by Permobil.

## United States

### All Pro

John Monroe  
johnmonroe1@aol.com  
Maryland - Virginia

### Brewis Group Inc.

Barry Brewis  
barryb@brewisgroup.com  
Portland - Oregon - Southwest Washington

Wayne Grapes  
wayne@brewisgroup.com  
California North Central - Nevada

Ken Nelson  
kenn@brewisgroup.com  
Alaska - Oregon

Duane Rhodes  
duaner@brewisgroup.com  
Western Washington

Craig Richardson  
craigr@brewisgroup.com  
California Bay

Lisa Vons Cooper  
lisav@brewisgroup.com  
Los Angeles California

Brad White  
bradw@brewisgroup.com  
Southern California

Ralph Zimmerman  
ralphz@brewisgroup.com  
Idaho - Montana - East Washington

### Ed Med

Ed Brooks  
edbrooks3@verizon.net  
Pennsylvania East - New Jersey

### Rehab Concepts

Mike Carlson  
mikec@rehabconcepts.org  
Minnesota - Iowa - North Dakota - South Dakota

### Rehab Marketing

Melissa Coombs  
melissa@rehabmarketing.com  
Maine - Vermont - New Hampshire

Richard Dove  
richard@rehabmarketing.com  
Massachusetts - Connecticut - Rhode Island

### Rehab Representatives

Paula Maiberger  
pmaiberger@mindspring.com  
Alabama - Georgia

### Roger Felske Associates Inc

Roger Felske  
rfelske@aol.com  
New York

### Southern Rehab Reps

Lynn Springfield  
southernrehabrep@gmail.com  
Central Texas - South Texas -  
- Louisiana - Mississippi

Billy Good  
billygood@sbcglobal.net  
North Texas - Oklahoma - Arkansas

### Target Medical Sales

Bob Brown  
bob@targetmed.com  
North Ohio - West Pennsylvania - West Virginia

Casey Clark  
casey@targetmed.com  
Ohio - Michigan

Mark Swearingen  
mark@targetmed.com  
Indiana - Kentucky

### Townsend Rep Group

John Burley  
jburley@comcast.net  
Florida

Alicia Westbrook  
awestbrook@townsendrepgroup.com  
North Carolina - South Carolina  
Jacksonville Florida - Tennessee

### Willcare Associates

Willa Moats  
willcare@ptd.net  
New York

Joe Norton  
jnorton810@aol.com  
Delaware - New Jersey

## International

### Abilia

Bjorn Amtzen  
info@abilia.no  
Norway

### Avoris Oy

Jorma Kokko  
myynti@avoris.fi  
Finland

### BES Rehab Ltd.

Barend ter Haar BSc DPhil ATP  
barend@bescorporate.net  
United Kingdom  
**Boavista Solutions**  
Adner Silva  
adner@boavistasolutions.eu  
Portugal

### Bodytech S.r.l

Andrea De Ranieri  
info@bodytechitalia.it  
Italy

### Doove Products BV

Bas Roeleveld  
products@doove.nl  
Netherlands Holland

### Handico Finland

Sami Martin  
sami.martin@handico.fi  
Finland

### Invacare

Ian Mortimer  
sales@invacare.com.au  
Australia

### Charles Harrison

sales@invacare.ca  
Canada

Yvonne Glidden  
sales@invacare.co.nz  
New Zealand

### JC Medical Instruments

Mr. Kevin Wei  
kevin@jc-medical.com  
Taiwan

## International (CONT)

### Licon Rehab

Mr. Gordon Wong  
liconrehab@ctimail3.com  
Hong Kong, China

### Madame Dubarry, S.A. de C.V.

Karla Plant  
kplant@medical-center.com.mx  
Mexico

### Medema Norge

Peter Nielsen  
firmapost@medema.no  
Norway, Denmark, Sweden

### Medimec

Dr. Mathias Capelli  
customerservice@medimec.it  
Italy

### MMS

Jessica Greaney  
info@mmsmedical.ie  
Ireland

### Mobility One

Donovan Walters  
donovan@mobilityone.co.za  
South Africa

### Rehab®

Diego Rios  
ventas@ortopediarehab.com.ar  
South America, Colombia, Argentina, Uruguay

### Rochester

Eliseo Espinas Salas  
principal@rochestertermexico.com  
Mexico

### Rubato Co., LTD

James  
rubatocare@yahoo.com  
South Korea

### Rupiani

Jean-Marie Mangenot - President  
jm.mangenot@rupiani.fr  
France

### SIP Seating SL

Filipe Correia  
info@sipseating.com  
Spain

### Special Needs For Special People

Yishak  
yishak@netvision.net.il  
Israel

### Technik & Reha Vertriebs GmbH

Albert Föhrenbühler  
a.föhrenbühler@t-rv.de  
Germany

### Technoline Ltd.

Charlot Scicluna  
cscicluna@technoline-mt.com  
Malta

### Tendercare Ltd

Amanda Adams  
info@tendercareltd.com  
United Kingdom, Ireland

### Unique Perspectives

Brian Dillon  
brian@click2go.ie  
Ireland

### Yuki Trading

Mr. Koji Sano  
sano@yukitrading.com  
Japan

## Home Office TEAM MEMBERS

### Management

#### Lorenzo Romero

CEO  
lorenzo@stealthproducts.com

#### Cesar Perea

Director of Manufacturing  
cesar@stealthproducts.com

#### Gabriel Romero

Director of Sales and Marketing  
gabriel@stealthproducts.com

#### Mark Scott

National Sales Manager  
mark@stealthproducts.com

#### Cory Monks

Seating Manager  
cory@stealthproducts.com

#### Scott Brown

Business Development Manager  
scott@stealthproducts.com

#### Mitch Komisar

Global Sales Manager  
mitch@stealthproducts.com

#### Karen Dowers

Sales Manager  
karen@stealthproducts.com

### Accounting

#### Marietta C. Williams

Accounts Receivables  
marietta@stealthproducts.com

### Seating Specialists

#### Donna Yamamoto

Custom Seating Representative  
donna@stealthproducts.com

#### Janice Garner

Custom Seating Specialist  
janice@stealthproducts.com

#### Shanon R. Davis

Custom Seating Specialist  
shannon@stealthproducts.com

#### Trey Beninghove

Custom Seating Representative  
trey@stealthproducts.com

#### Vanessa Hoover

Custom Seating Representative  
vanessa@stealthproducts.com

### Customer Service

#### Amy Kafka

Customer Service Representative  
amy@stealthproducts.com

#### Gia Ming

Custom Seating Representative  
gia@stealthproducts.com

#### Melissa Barnard

Customer Service Representative  
melissa@stealthproducts.com

#### Molly Warnock

Customer Service Representative  
molly@stealthproducts.com

#### Velma Beltran

Customer Service Representative  
velma@stealthproducts.com

### International Representatives

#### Katrina Scott

International Representative  
katrina@stealthproducts.com

The following States are being served directly by our home office

ARIZONA • COLORADO • ILLINOIS • KANSAS • MISSOURI • NEBRASKA • NEW MEXICO • UTAH • VERMONT • WYOMING



# And Growing...

Become a distributor and be part of our team.



1.800.965.9229 | fx 1.800.806.1225 | info@stealthproducts.com | 104 John Kelly Drive, Burnet, TX 78611